

Patient Information

Who is ETS Wound Care?

ETS Wound Care is an emerging wound care company focused on using ETS's borate-based bioactive technology to develop and commercialize a broad range of breakthrough solutions for the wound care market to minimize the cost of treatment, while improving patient outcomes. Learn more at www.etswoundcare.com.



4030 Hypoint North • Rolla, MO 65401

O: 573.202.2550 • F: 573.755.0588

etswoundcare.com

MKT-0007 REV 1

© 2018 ETS Wound Care, LLC

MIRRAGEN® is a registered trademark of ETS Tech Holdings, LLC




Your guide to understanding
MIRRAGEN® Advanced Wound Matrix
and its role in your treatment

MIRRAGEN®

ADVANCED WOUND MATRIX

What is MIRRAGEN® Advanced Wound Matrix?

MIRRAGEN® Advanced Wound Matrix is a fibrous material designed for the treatment of wounds. The material is composed of a unique borate-based glass that is 100% absorbable by the body and resembles cotton candy in appearance.

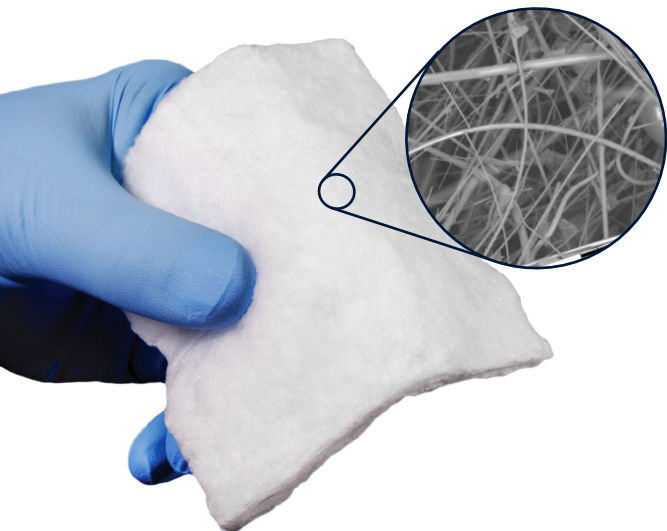

How does it work?

The fibrous structure of MIRRAGEN® allows it to wick fluid away from the wound bed and control the moisture content to facilitate the wound healing response. During the healing process, MIRRAGEN® is absorbed by the surrounding tissue.

Is it safe?

Yes. MIRRAGEN® Advanced Wound Matrix has been cleared by the United States FDA. Unlike some other wound care products, MIRRAGEN® does not contain any biologic based raw materials.

How will it be used to treat my wound?

MIRRAGEN® is typically applied 1–2 times per week, depending on the nature of the wound, rate of healing, and other factors.

At the beginning of each visit, your wound care professional will clean and examine your wound, and remove any dead tissue if necessary.

MIRRAGEN® will then be applied directly to the wound, covering all exposed tissue. A secondary dressing will be applied to hold MIRRAGEN® in place and protect the wound environment until your next visit.

